



2. sew the two parts together along the curved edge, face to face with elastic inside. Snip small Vs into seam edge to allow for folds.

1. on both parts: stitch elastic ear loops into place on the face (exterior) of the fabric

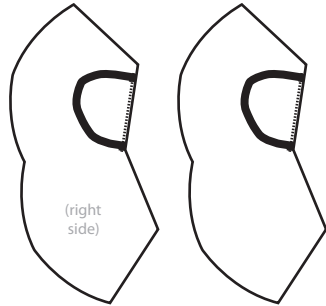

3. On bottom hems: fold at 3/8" and hem. This will be the filter opening.

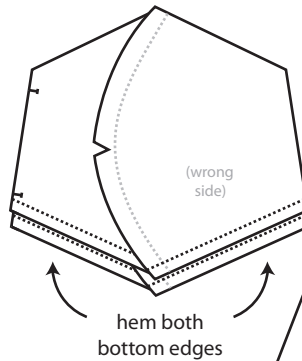


TOM BIHN MASK v3 (adapted) with bottom-access filter pocket

- pattern: print at 100%, cut out
- fabric: cut two pieces per mask
- elastic: cut two pieces 7.5" (190mm) long
- nose bridge (optional): .5-.6" (12.5-15mm) wide tape cut to 4.25" (108mm) long
- bridge wire (optional metal strip, pipe cleaner, wire, etc.): cut to ~4" (100mm)

For more information on mask making, including videos, instructions for use and warnings, see <https://www.tombihn.com/blogs/main/maskhowto>

(note: this is a fan remix, not an official Tom Bihn pattern!)

4. Fold the mask along its upper edge, inverting the two halves (face sides and elastic should be tucked inside). Stitch both sides, leaving the bottom center open. Be careful to stitch just the ends of the elastic.

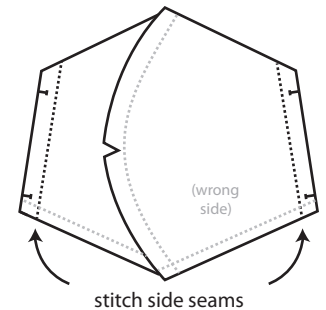

5. Right side out the mask and top-stitch around sides, top and an inch on either side of the bottom. Add optional fabric for nose bridge or stitch 4.25" seam at center top, then insert wire from the inside. You're done!

